A. GEBRUIK

De hier vermelde handschoentypen voldoen aan de vereisten in de verordening (EU) 2016/425, EN ISO 374-1:2016+A1:2018 en EN ISO 21420:2020.

EN ISO 374-1 RESULTATEN VAN CHEMISCHE TESTS table with columns for test type, chemical, level, and result.

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Table mapping test types (A-E) to chemical and microorganism resistance levels.

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Table showing breakthrough time (min) for different levels (1-6) and test types.

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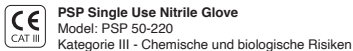
REGULATION (EU) 2016/425, EN 455 MEDICAL DEVICE, and size selection table.

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GENERAL EN ISO 374 TESTING PERFORMANCE REQUIREMENTS:

Table mapping test types to chemical and microorganism resistance levels for DE section.

Definition of breakthrough time through the glove palm


Table showing breakthrough time (min) for different levels and test types for DE section.

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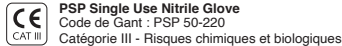REGULATION (EU) 2016/425, EN 455 MEDICAL DEVICE, and size selection table for DE section.

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Table mapping test types to chemical and microorganism resistance levels for FR section.

Definition of breakthrough time through the glove palm

Table showing breakthrough time (min) for different levels and test types for FR section.

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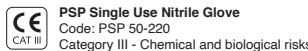
REGULATION (EU) 2016/425, EN 455 MEDICAL DEVICE, and size selection table for FR section.

A. USE

The use types named here meet with the requirements of regulation (EU) 2016/425, EN ISO 374-1:2016+A1:2018 and EN ISO 21420:2020.

RESULTS CHEMICAL TESTS: Table with test results for EN section.

GENERAL EN ISO 374 TESTING PERFORMANCE REQUIREMENTS:

Table mapping test types to chemical and microorganism resistance levels for EN section.

Definition of breakthrough time through the glove palm

Table showing breakthrough time (min) for different levels and test types for EN section.

B. WARNING / RISK ASSESSMENT:

a) Gloves don't provide the protection against mechanical action whose effects are superficial, substances and mixtures which are hazardous to health...

USE - Donning: hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger at a time.

REGULATION (EU) 2016/425, EN 455 MEDICAL DEVICE, and size selection table for EN section.